


Procedure Steps on One Page for Quick Reference



STEP - 1

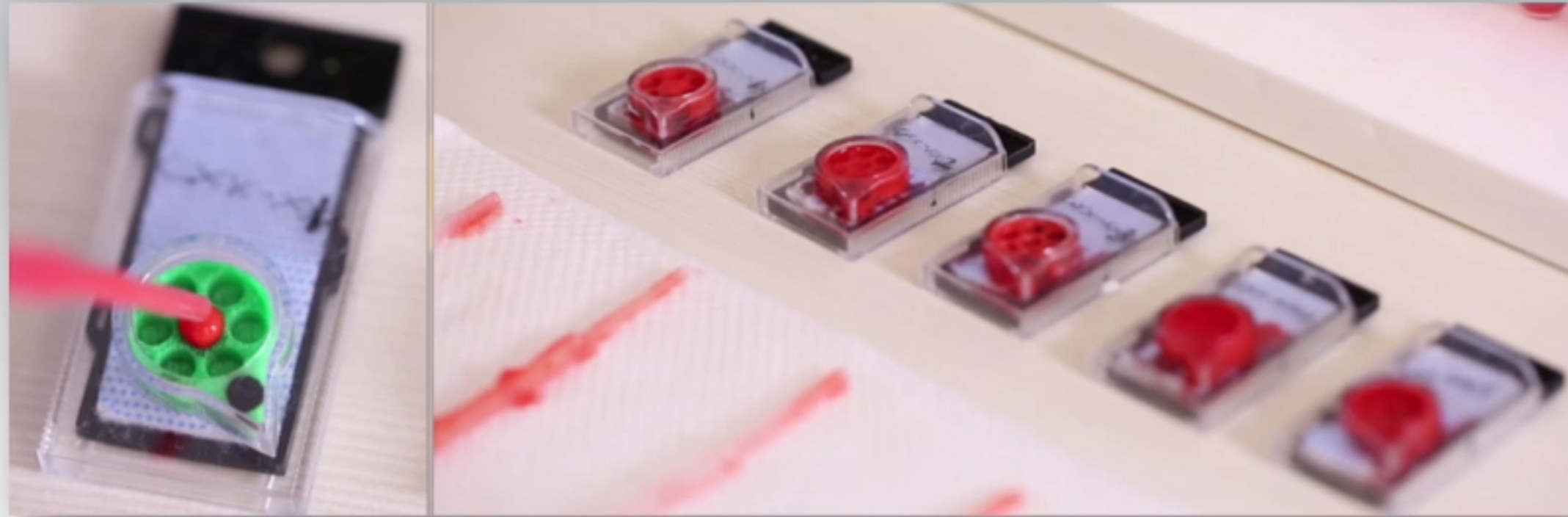

STEP - 2



Add concentrated specimens on the respective sponge discs and keep the respective transfer pipettes in their places for use at later stage to dislodge the respective sponge discs with concentrated specimen into the respective tissue cassettes

Add 10% formalin fixative to all sponge discs with concentrated specimen

STEP - 5



Let all the cassette fix in horizontal position with bottom down for more than 2 hours. If cross-contamination is anticipated, fix and transport each tissue cassettes individually (instead of stacking and transporting multiple tissue cassettes in one container shown in this demo).

STEP - 4



Put all the sponge discs with concentrated sediments and covered with tissue paper cover between two tissue sponges Into their respective tissue cassettes

STEP - 3



Transfer the sponge discs with the tip of transfer pipette in to their tissue cassette